

Cochlear™ Baha® 5 Systems

Hearing success begins with the implant.


With its introduction in 2010, the Cochlear™ Baha® B1300 Implant set the industry standard for reliability and performance. Backed by clinical research demonstrating excellent implant survivability,¹⁻⁷ the B1300 provides a strong foundation for both the Baha Attract and Baha Connect System.

The proven direct system

The clinical performance of the Baha Connect System has been proven through the largest^{8,9} prospective study in bone conduction. The system features unique Dermalock™ technology proven to integrate with soft tissue.¹⁰



Baha Connect System

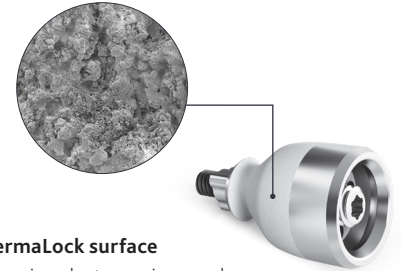


Dermalock surface

Scanning electron micrograph of the Dermalock surface.

Proven soft tissue integration

Minimally invasive surgery with Dermalock technology promotes soft tissue integration.^{10,11} In a clinical study, the Baha BA400 (Dermalock) Abutment demonstrates a statistically significant reduction in numbness, faster wound healing, less neuropathic pain, improved aesthetic outcomes and fewer abutment changes due to skin overgrowth.^{8,9}

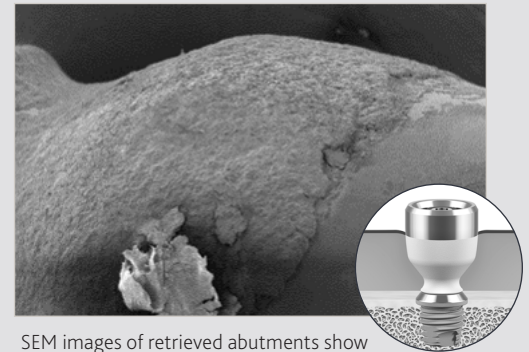

SEM images of retrieved abutments show that only the Dermalock surface (unlike all-titanium abutments) is capable of binding with viable soft tissue.³

The most complete sound processor portfolio


The Baha Connect System's head-worn power options deliver access to more gain through higher power output than any other bone conduction implant system.¹²

Excellent soft tissue outcomes

The performance of the Dermalock surface has been confirmed through a prospective study. Of the 30 patients included, 93% reported no or mild soft tissue reactions (Holgers grade 0-1).¹³



Excellent soft tissue outcomes, 93% mild or no soft tissue reactions

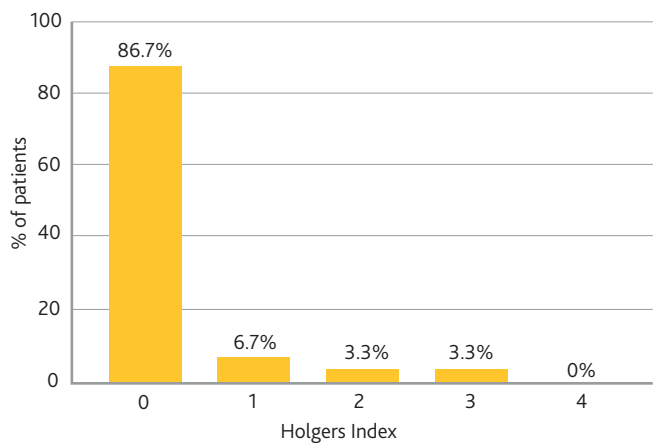

Hear now. And always



The most powerful under-the-skin system.¹²


Since its introduction in August 2013, the Baha Attract System is the most widely used under-the-skin bone conduction system in the industry.

More high frequency power

The Baha Attract System delivers more power at high frequencies with both Baha 5 Power and Baha 5 SuperPower compared with the previous generation power device. It can be fitted with a choice of three head-worn sound processors.



Baha Attract System


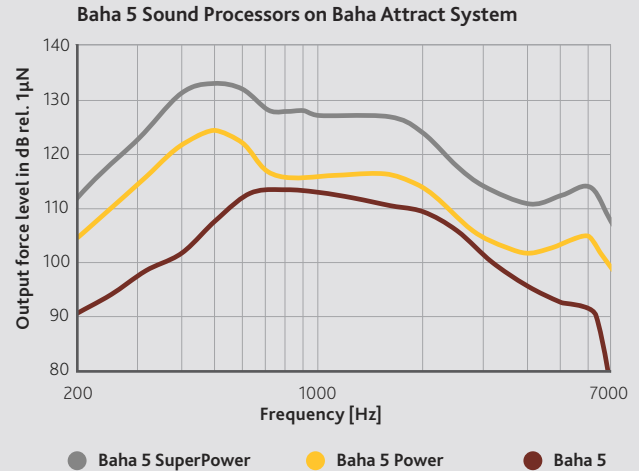


OFL90 Baha Attract System measured on Artificial Mastoid IEC 60318-6.^{12,14-16}

Significantly improved hearing outcomes

Results from a multi-center clinical study demonstrate significantly improved outcomes for patients with mixed hearing loss using the Baha Attract System with a Baha 5 SuperPower.¹⁷

Significantly improved APHAB scores when compared to the unaided situation (Global, Ease of communication, Reverberation and Background noise).

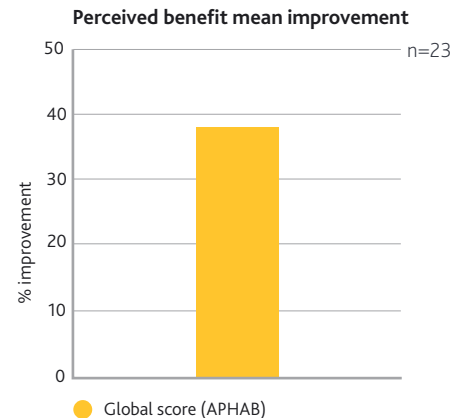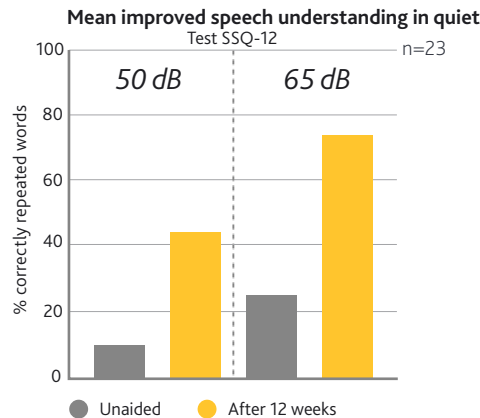

- den Besten CA, Stalfors J, Wigren S, Blechert JI, Flynn M, Eeg-Olofsson M, Aggarwal R, Green K, Nelissen RC, Mylanus EA, Hol MK. Stability, Survival, and Tolerability of an Auditory Osseointegrated Implant for Bone Conduction Hearing: Long-Term Follow-Up of a Randomized Controlled Trial. *Otol Neurotol*. 2016;37(8):1077-1083.
- D'Eredita R, Caroncin M, Saetti R. The new Baha implant: a prospective osseointegration study. *Otolaryngol Head Neck Surg*. 2012;146(6):979-983.
- Felton M, Hill-Feltham P, Bruce IA. The role of stability measurements of the Baha(R) system in children. *Int J Pediatr Otorhinolaryngol*. 2014;78(3):513-516.
- Hussemann J, Szudek J, Monksfield P, Power D, O'Leary S, Briggs R. Simplified bone-anchored hearing aid insertion using a linear incision without soft tissue reduction. *J Laryngol Otol*. 2013;127 Suppl 2:S33-38.
- Marsella P, Scorpecci A, D'Eredita R, Della Volpe A, Malerba P. Stability of osseointegrated bone conduction systems in children: a pilot study. *Otol Neurotol*. 2012;33(5):797-803.
- Wilkie MD, Chakravarthy KM, Mamais C, Temple RH. Osseointegrated hearing implant surgery using a novel hydroxyapatite-coated concave abutment design. *Otolaryngol Head Neck Surg*. 2014;151(6):1014-1019.
- Wilkie MD, Lightbody KA, Salamat AA, Chakravarthy KM, Luff DA, Temple RH. Stability and survival of bone anchored hearing aid implant systems in post-irradiated patients. *Eur Arch Otorhinolaryngol*. 2015;272(6):1371-1376.
- Clinical and health economic evaluation with a new Baha abutment combined with a minimally invasive surgical technique. *ClinicalTrials.gov* identifier: NCT01796236. Cochlear Bone Anchored Solutions AB, Mölnlycke, Sweden.
- van Hoof M, et al. A RCT of soft tissue preservation using a HA-coated abutment in percutaneous bone conduction hearing implant surgery –1-year clinical outcomes. Paper presented at: 14th International Conference on Cochlear Implants and Other Implantable Technologies; May, 2016; Toronto, Canada.
- van Hoof M, et al. (2015) Can the Hydroxyapatite-Coated Skin-Penetrating Abutment for Bone Conduction Hearing Implants Integrate with the Surrounding Skin? *Front Surg*.2:45.
- Kapsokalyvas D, et al. (2017) Investigating the race for the surface and skin integration in clinically retrieved abutments with two-photon microscopy. *Colloids Surf B Biointerfaces*.159:97-107.
- Norrman J. (2015) Review of fitting ranges. Cochlear Bone Anchored Solutions AB, D773528.
- den Besten CA, et al. (2015) Clinical results of Cochlear BIA300 in children: Experience in two tertiary referral centers. *Int J Pediatr Otorhinolaryngol*.79(12):2050-2055.
- 630909-4, Cochlear Baha 5 datasheet.
- D801287, Cochlear Baha 5 Power datasheet
- D770056, Cochlear Baha 5 SuperPower datasheet
- Ray J. Evaluation of audiological outcomes and subjective benefits of Cochlear Baha 5 SuperPower Sound Processor on the Baha Attract System. Paper presented at: Osseo2017; Nijmegen, NL.

©Cochlear Limited 2018. All rights reserved. Hear now. And always and other trademarks and registered trademarks are the property of Cochlear Limited or Cochlear Bone Anchored Solutions AB. The names of actual companies and products mentioned herein may be the trademarks of their respective owners.